

## Overview

### Method Name

Fluoroimmunoassay; Luminex Multiplex platform

### NY State Available

Yes

## Specimen

### Specimen Type

Serum

### Specimen Required

Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5mL of serum refrigerated in a plastic vial.

### Reject Due To

Hemolysis: NA

Thawing: Warm OK; Cold OK

Lipemia: NA

Icterus: NA

Other: NA

### Specimen Minimum Volume

0.5 mL

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	7 days	
	Frozen	365 days	
	Ambient	7 days	

## Clinical & Interpretive

### Reference Values

Pneumococcal Antibody Panel Reference Range

Pneumo Ab Type 1 >1.3 ug/mL

Pneumo Ab Type 3 >1.3 ug/mL

Pneumo Ab Type 4 >1.3 ug/mL

Pneumo Ab Type 8 >1.3 ug/mL

---

Pneumo Ab Type 9 (9N) >1.3 ug/mL  
Pneumo Ab Type 12 (12F) >1.3 ug/mL  
Pneumo Ab Type 14 >1.3 ug/mL  
Pneumo Ab Type 19 (19F) >1.3 ug/mL  
Pneumo Ab Type 23 (23F) >1.3 uh/mL  
Pneumo Ab Type 26 (6B) >1.3 ug/mL  
Pneumo Ab Type 51 (7F) >1.3 ug/mL  
Pneumo Ab Type 56 (18C) >1.3 ug/mL

## Performance

### PDF Report

No

### Performing Laboratory Location

Eurofins Viracor

## Fees & Codes

### Test Classification

[This test was developed and its performance characteristics determined by Viracor Eurofins. It has not been cleared or approved by the U.S. Food and Drug Administration.](#)

### CPT Code Information

86317 x 12