

## Overview

### Method Name

Enzyme Immunoassay (FEIA)

### NY State Available

No

## Specimen

### Specimen Type

Serum

### Specimen Required

Draw blood in a plain, red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.

### Reject Due To

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	NA

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Frozen	365 days	
	Ambient	7 days	

## Clinical & Interpretive

### Reference Values

Reference ranges have not been established for food-specific IgG tests. The clinical utility of food-specific IgG tests has not been established. These tests can be used in special clinical situations to select foods for evaluation by diet elimination and challenge in patients who have food-related complaints. It should be recognized that the presence of food-specific IgG alone cannot be taken as evidence of food allergy and only indicates immunologic sensitization by the food allergen in question. This test should only be ordered by physicians who recognize the limitations of the test.

**Performance****PDF Report**

No

**Performing Laboratory Location**

Eurofins Viracor

**Fees & Codes****Test Classification**

[This test was developed and its performance characteristics determined by Viracor Eurofins. It has not been cleared or approved by the U.S. Food and Drug Administration.](#)

**CPT Code Information**

86001

**LOINC® Information**

Test ID	Test Order Name	Order LOINC Value
FONG	Onion IgG	35541-2

Result ID	Reporting Name	LOINC®
FONG	Onion IgG	35541-2