

## Overview

### Method Name

Immunoassay (IA)

Liquid Chromatography/Tandem Mass Spectrometry (LC/MS/MS)

### NY State Available

Yes

## Specimen

### Specimen Type

Varies

### Specimen Required

**Submit only 1 of the following:**

#### Plasma

Draw blood in a green-top (sodium heparin) tube(s), **plasma gel tube is not acceptable**. Spin down and send 3 mL of plasma refrigerated in a plastic vial.

#### Serum

Draw blood in a plain red-top tube(s), **serum gel tube is not acceptable**. Spin down and send 3 mL of serum refrigerated in a plastic vial,

#### Urine

Collect 3 mL random urine and send refrigerated in a preservative free plastic urine container.

### Reject Due To

Hemolysis NA

Lipemia NA

Icterus NA

Other NA

### Specimen Minimum Volume

0.6 mL

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	7 days	
	Frozen	180 days	
	Ambient		

## Clinical & Interpretive

**Reference Values**

Antidepressant screen includes the analysis for:

Amytriptyline, Clomipramine and Desmethylclomipramine, Cyclobenzaprine, Desipramine, Doxepin and Desmethyldoxepin, Fluoxetine and Norfluoxetine, Imipramine, Maprotiline, Nortriptyline, Paroxetine, Protriptyline, Sertraline and Desmethylsertraline, Trimipramine.

**Performance****PDF Report**

No

**Performing Laboratory Location**

Medtox Laboratories, Inc.

**Fees & Codes****Test Classification****CPT Code Information**

80333

80337

80369