

## Overview

### Testing Algorithm

If FOVAS "Ovarian Ab Screen w/Reflex" is positive, then FOVAS "Anti-Ovary Ab Titer" will be performed at an additional charge.

### Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
FOVAS	Anti-Ovary Ab Titer	No	No

### Method Name

Immunofluorescence Assay (IFA)

### NY State Available

Yes

## Specimen

### Specimen Type

Serum

### Specimen Required

Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 1 mL of serum refrigerated in a plastic vial.

### Reject Due To

Hemolysis Mild OK; Gross reject  
 Lipemia Mild OK; Gross reject  
 Icterus NA  
 Other NA

### Specimen Minimum Volume

0.3 mL

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	14 days	
	Frozen	30 days	
	Ambient	7 days	

## Clinical & Interpretive

## Reference Values

Negative

Ovarian antibody screen is at 1:5 dilution

## Performance

### PDF Report

No

### Performing Laboratory Location

Quest Diagnostics Nichols Institute

## Fees & Codes

### Test Classification

#### CPT Code Information

86255

86256 (if appropriate)