

Overview**Method Name**

ImmunoCAP FEIA

NY State Available

Yes

Specimen**Specimen Type**

Serum

Specimen Required

Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.

Reject Due To

Hemolysis	Mild OK; Gross reject
Lipemia	Mild OK; Gross reject
Icterus	Mild OK; Gross reject
Other	NA

Specimen Minimum Volume

0.5 mL

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Frozen	365 days	
	Ambient	28 days	

Clinical & Interpretive**Reference Values**

<0.35 kU/L

Interpretation

<u>Class</u>	<u>IgE (kU/L)</u>	<u>Comment</u>
0	<0.10	Negative
0/1	0.10 â€” 0.34	Equivocal/Borderline
1	0.35 â€” 0.69	Low Positive
2	0.70 â€” 3.49	Moderate Positive
3	3.50 â€” 17.49	High Positive
4	17.50 â€” 49.99	Very High Positive
5	50.00 â€” 99.99	Very High Positive
6	>99.99	Very High Positive

Performance**PDF Report**

No

Performing Laboratory Location

Eurofins Viracor

Fees & Codes**Test Classification**

This test was developed and its performance characteristics determined by Viracor Eurofins. It has not been cleared or approved by the U.S. Food and Drug Administration.

CPT Code Information

86003

LOINC® Information

Test ID	Test Order Name	Order LOINC Value
FLNDR	Flounder IgE	7316-3

Result ID	Reporting Name	LOINC®
Z3815	Flounder IgE	7316-3
Z3816	CLASS	15715-6