

## Overview

### Method Name

ImmunoCAP FEIA

### NY State Available

Yes

## Specimen

### Specimen Type

Serum

### Specimen Required

Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.

### Reject Due To

Hemolysis: Mild OK; Gross reject  
Thawing: Warm OK; Cold OK  
Lipemia: Mild OK; Gross reject  
Icterus: Mild OK; Gross reject  
Other: NA

### Specimen Minimum Volume

0.5 mL

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Frozen	365 days	
	Ambient	28 days	

## Clinical & Interpretive

### Reference Values

---

<0.35 kU/L

**Interpretation**

<u>Class</u>	<u>IgE (kU/L)</u>	<u>Comment</u>
0	<0.10	Negative
0/1	0.10 – 0.34	Equivocal/Borderline
1	0.35 – 0.69	Low Positive
2	0.70 – 3.49	Moderate Positive
3	3.50 – 17.49	High Positive
4	17.50 – 49.99	Very High Positive
5	50.00 – 99.99	Very High Positive
6	>99.99	Very High Positive

**Performance****PDF Report**

No

**Performing Laboratory Location**

Eurofins Viracor

**Fees & Codes****Test Classification**

This test was developed and its performance characteristics determined by Viracor Eurofins. It has not been cleared or approved by the U.S. Food and Drug Administration.

**CPT Code Information**

86003

---

**LOINC® Information**

Test ID	Test Order Name	Order LOINC Value
FIGE	Fig IgE	11173-2

Result ID	Reporting Name	LOINC®
Z3870	Fig IgE	11173-2
Z3871	CLASS	21270-4