

**Overview****Method Name**

ImmunoCAP FEIA

**NY State Available**

Yes

**Specimen****Specimen Type**

Serum

**Specimen Required**

Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.

**Specimen Minimum Volume**

0.5 mL

**Reject Due To**

Hemolysis	Mild OK; Gross reject
Lipemia	Mild OK; Gross reject
Icterus	Mild OK; Gross reject
Other	NA

**Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Frozen	365 days	
	Ambient	28 days	

**Clinical & Interpretive****Reference Values**

&lt;0.35 kU/L

**Interpretation**

<u>Class</u>	<u>IgE (kU/L)</u>	<u>Comment</u>
0	<0.10	Negative
0/1	0.10 – 0.34	Equivocal/Borderline
1	0.35 – 0.69	Low Positive
2	0.70 – 3.49	Moderate Positive
3	3.50 – 17.49	High Positive
4	17.50 – 49.99	Very High Positive
5	50.00 – 99.99	Very High Positive
6	>99.99	Very High Positive

## Performance

### PDF Report

No

### Day(s) Performed

Monday through Friday

### Report Available

1 to 6 days

### Performing Laboratory Location

Eurofins Viracor

## Fees & Codes

### Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

### CPT Code Information

86003

### LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FNBE	Bean Navy/White IgE	6279-4

Result ID	Test Result Name	Result LOINC® Value
Z3992	Bean Navy/White IgE	6279-4

---

Z3993	CLASS	15569-7
-------	-------	---------