

Overview**Method Name**

ImmunoCAP FEIA

NY State Available

Yes

Specimen**Specimen Type**

Serum

Specimen Required

Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.

Reject Due To

Hemolysis Mild OK; Gross reject

Lipemia Mild OK; Gross reject

Icterus Mild OK; Gross reject

Other NA

Specimen Minimum Volume

0.5 mL

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Frozen	365 days	
	Ambient	28 days	

Clinical & Interpretive**Reference Values**

<0.35 kU/L

Interpretation

Class	IgE (kU/L)	Comment
0	<0.10	Negative

0/1	0.10-0.34	Equivocal/Borderline
1	0.35-0.69	Low Positive
2	0.70-3.49	Moderate Positive
3	3.50-17.49	High Positive
4 5 6	17.50-49.99 50.0-99.99 >99.99	Very High Positive Very High Positive Very High Positive

Performance

PDF Report

No

Performing Laboratory Location

Eurofins Viracor

Fees & Codes

Test Classification

[This test was developed and its performance characteristics determined by Viracor Eurofins.](#) It has not been cleared or approved by the U.S. Food and Drug Administration.

CPT Code Information

86003