

Overview

Method Name

ImmunoCAP FEIA

NY State Available

Yes

Specimen

Specimen Type

Serum

Specimen Required

Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.

Reject Due To

Hemolysis Mild OK; Gross reject

Lipemia Mild OK; Gross reject

Icterus Mild OK; Gross reject

Other NA

Specimen Minimum Volume

0.5 mL

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Frozen	365 days	
	Ambient	28 days	

Clinical & Interpretive

Reference Values

<0.35 kU/L

InterpretationClass IgE (kU/L) Comment

0 <0.10 Negative

0/1 0.10 – 0.34 Equivocal/Borderline

-
- 1 0.35 – 0.69 Low Positive
 - 2 0.70 – 3.49 Moderate Positive
 - 3 3.50 – 17.49 High Positive
 - 4 17.50 – 49.99 Very High Positive
 - 5 50.00 – 99.99 Very High Positive
 - 6 >99.99 Very High Positive

Performance

PDF Report

No

Performing Laboratory Location

Eurofins Viracor

Fees & Codes

Test Classification

CPT Code Information

86003