

Overview**Method Name**

ImmunoCAP FEIA

NY State Available

Yes

Specimen**Specimen Type**

Serum

Specimen Required

Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.

Reject Due To

Hemolysis	Mild OK; Gross reject
Lipemia	Mild OK; Gross reject
Icterus	Mild OK; Gross reject
Other	NA

Specimen Minimum Volume

0.5 mL

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Frozen	365 days	
	Ambient	28 days	

Clinical & Interpretive**Reference Values**

<0.35 kU/L

Interpretation

<u>Class</u>	<u>IgE (kU/L)</u>	<u>Comment</u>
0	<0.10	Negative
0/1	0.10 – 0.34	Equivocal/Borderline
1	0.35 – 0.69	Low Positive
2	0.70 – 3.49	Moderate Positive
3	3.50 – 17.49	High Positive
4	17.50 – 49.99	Very High Positive
5	50.00 – 99.99	Very High Positive
6	>99.99	Very High Positive

Performance**PDF Report**

No

Performing Laboratory Location

Eurofins Viracor

Fees & Codes**Test Classification****CPT Code Information**

86003

LOINC® Information

Test ID	Test Order Name	Order LOINC Value
---------	-----------------	-------------------

FPRTF	Parrot Australian Feathers IgE	6030-1
Result ID	Reporting Name	LOINC®
Z4052	Parrot Australian Feathers IgE	6030-1
Z4053	CLASS	15592-9