

Overview**Method Name**

ImmunoCAP FEIA

NY State Available

Yes

Specimen**Specimen Type**

Serum

Specimen Required

Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.

Specimen Minimum Volume

0.5 mL

Reject Due To

| | |
|-----------|-----------------------|
| Hemolysis | Mild OK; Gross reject |
| Lipemia | Mild OK; Gross reject |
| Icterus | Mild OK; Gross reject |
| Other | NA |

Specimen Stability Information

| Specimen Type | Temperature | Time | Special Container |
|---------------|--------------------------|----------|-------------------|
| Serum | Refrigerated (preferred) | 28 days | |
| | Frozen | 365 days | |
| | Ambient | 28 days | |

Clinical & Interpretive**Reference Values**

<0.35 kU/L

Interpretation

| <u>Class</u> | <u>IgE (kU/L)</u> | <u>Comment</u> |
|--------------|-------------------|----------------------|
| 0 | <0.10 | Negative |
| 0/1 | 0.10 – 0.34 | Equivocal/Borderline |
| 1 | 0.35 – 0.69 | Low Positive |
| 2 | 0.70 – 3.49 | Moderate Positive |
| 3 | 3.50 – 17.49 | High Positive |
| 4 | 17.50 – 49.99 | Very High Positive |
| 5 | 50.00 – 99.99 | Very High Positive |
| 6 | >99.99 | Very High Positive |

Performance**PDF Report**

No

Day(s) Performed

Monday through Friday

Report Available

1 to 6 days

Performing Laboratory Location

Eurofins Viracor

Fees & Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their Regional Manager. For assistance, contact [Customer Service](#).

Test Classification

[This test was developed and its performance characteristics determined by Viracor Eurofins. It has not been cleared or approved by the U.S. Food and Drug Administration.](#)

CPT Code Information

86003

LOINC® Information

| Test ID | Test Order Name | Order LOINC® Value |
|---------|-----------------|--------------------|
|---------|-----------------|--------------------|

| | | |
|-------|-----------------------------------|--------|
| FOAKE | Oak Live (Quercus virginiana) IgE | 6164-8 |
|-------|-----------------------------------|--------|

| Result ID | Test Result Name | Result LOINC® Value |
|-----------|-----------------------------------|---------------------|
| Z4118 | Oak Live (Quercus virginiana) IgE | 6164-8 |
| Z4119 | CLASS | 15880-8 |