

## Overview

### Testing Algorithm

If the GHB Screen S/P is non-negative, then the Gamma-Hydroxybutyric Acid, CF, SP (FGHA) will be performed at an additional charge.

### Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
FGHA	Gamma-Hydroxybutyric Acid, CF, SP	No	No

### Method Name

Liquid Chromatography with Tandem Mass Spectrometry (LC/MS/MS)

Gas Chromatography/Mass Spectrometry (GC/MS) (if appropriate)

### NY State Available

Yes

## Specimen

### Specimen Type

Varies

### Specimen Required

Submit only 1 of the following specimens:

#### Serum

Draw blood in a plain red-top tube(s), **serum gel tube is not acceptable**. Spin down and send 5 mL of serum refrigerated in a plastic vial.

#### Plasma

Draw blood in a green-top (sodium heparin) tube(s), **plasma gel tube is not acceptable**. Spin down and send 5 mL of sodium heparin plasma refrigerated in a plastic vial.

**Reject Due To**

Hemolysis NA  
Lipemia NA  
Icterus NA  
Other NA

**Specimen Minimum Volume**

1.2 mL

**Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	7 days	
	Frozen	180 days	
	Ambient		

**Clinical & Interpretive****Reference Values**

Reference Range: Negative

Screening threshold: 5.0 ug/mL

**Performance****PDF Report**

No

**Performing Laboratory Location**

Medtox Laboratories, Inc.

**Fees & Codes**

**Test Classification****CPT Code Information**

80307

**LOINC® Information**

Test ID	Test Order Name	Order LOINC Value
FGHSP	GHB Screen, S/P	46085-7

Result ID	Reporting Name	LOINC®
Z4223	GHB	46085-7