

## Overview

### Useful For

Aiding in the diagnosis of collagenofibrotic glomerulopathy

### Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
MLCPC	Microdissection, Laser Capture	No, (Bill Only)	No
MSPTC	Mass Spectrometry	No, (Bill Only)	No

### Method Name

Liquid Chromatography Tandem Mass Spectrometry (LC-MS/MS)

### NY State Available

Yes

## Specimen

### Specimen Type

Special

### Necessary Information

Preliminary pathology report, history, and electron microscopy images are required.

### Specimen Required

**Supplies:** Pathology Packaging Kit (T554)

**Specimen Type:** Formalin-fixed, paraffin-embedded kidney tissue block

**Collection Instructions:** Do not send fixed tissue slides. Testing can only be done on paraffin-embedded tissue blocks.

### Reject Due To

Fixed tissue slides Wet/frozen tissue Cytological smears Nonformalin	Reject
---	--------

fixed tissue Nonparaffin embedded tissue	
---	--

**Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Special	Ambient (preferred)		
	Frozen		
	Refrigerated		

**Clinical & Interpretive**
**Clinical Information**

Collagenofibrotic glomerulopathy (CG), also called collagen type III glomerulopathy, is a rare kidney disease characterized by large amounts of atypical type III collagen fibrils in the mesangium and subendothelial space of renal glomeruli. Liquid chromatography tandem mass spectrometry performed on microdissected glomeruli from patients with CG demonstrates a unique proteomic profile. The presence of type III collagen, in the appropriate clinical and pathological context, can be useful to establish a diagnosis of CG.

**Reference Values**

An interpretive report will be provided.

**Interpretation**

An interpretation will be provided.

**Cautions**

No significant cautionary statements

**Clinical Reference**

- Chen X, Wang H, Xu W, Zhu J. Collagen type III glomerulopathy: Case report and review of the literature. *Clin Nephrol.* 2017;87 (2017)(1):39-46. doi:10.5414/CN108907
- Dong J, Wei H, Han M, Guan Y, Wu Y, Li H. Collagen type III glomerulopathy: A case report and review of 20 cases. *Exp Ther Med.* 2015;10(4):1445-1449. doi:10.3892/etm.2015.2695
- Kurien AA, Larsen CP, Cossey LN. Collagenofibrotic glomerulopathy. *Clin Kidney J.* 2015;8(5):543-547. doi:10.1093/ckj/sfv061
- Cohen AH. Collagen type III glomerulopathies. *Adv Chronic Kidney Dis.* 2012;19(2):101-106. doi:10.1053/j.ackd.2012.02.017
- Duggal R, Nada R, Rayat CS, Rane SU, Sakhuja V, Joshi K. Collagenofibrotic glomerulopathy-a review. *Clin Kidney J.* 2012;5(1):7-12. doi:10.1093/ndtplus/sfr144

**Performance****Method Description**

Affected areas are removed from paraffin-embedded tissues by laser microdissection. Protein digestion is performed, followed by liquid chromatography tandem mass spectrometry.(Unpublished Mayo method)

**PDF Report**

No

**Day(s) Performed**

Monday through Friday

**Report Available**

7 to 15 days

**Specimen Retention Time**

Until reported

**Performing Laboratory Location**

Rochester

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**Test Classification**

This test was developed, and its performance characteristics determined by Mayo Clinic in a manner consistent with CLIA requirements. This test has not been cleared or approved by the US Food and Drug Administration.

**CPT Code Information**

82542

88380

**LOINC® Information**

Test ID	Test Order Name	Order LOINC® Value
MSCG	CG Confirm, LC MS	65757-7

Result ID	Test Result Name	Result LOINC® Value
-----------	------------------	---------------------

615299	Interpretation	50595-8
615300	Participated in the Interpretation	No LOINC Needed
615301	Report electronically signed by	19139-5
615302	Material Received	81178-6
615303	Disclaimer	62364-5
615304	Case Number	80398-1
617019	Gross Description	22634-0
617020	Addendum	35265-8