

## Overview

### Useful For

Detection and quantification of oxycodone, oxymorphone, noroxycodone, and noroxymorphone in urine

### Special Instructions

- [Clinical Toxicology CPT Code Client Guidance](#)

### Method Name

Liquid Chromatography Tandem Mass Spectrometry (LC-MS/MS)

### NY State Available

Yes

## Specimen

### Specimen Type

Urine

### Ordering Guidance

For situations where chain of custody is required, a Chain-of-Custody Kit (T282) is available. For chain-of-custody testing, order OXYCX / Oxycodone with Metabolite Confirmation, Chain of Custody, Random, Urine.

Additional drug panels and specific requests are available; call 800-533-1710 or 507-266-5700.

### Additional Testing Requirements

If urine creatinine is required or adulteration of the specimen is suspected, the following test should also be ordered, ADULT / Adulterants Survey, Random, Urine.

### Specimen Required

**Supplies:** Sarstedt Aliquot Tube 5 mL (T914)

**Collection Container/Tube:** Plastic urine container

**Submission Container/Tube:** Plastic, 5-mL tube

**Specimen Volume:** 3 mL

#### Collection Instructions:

1. Collect a random urine specimen.
2. No preservative.

#### Additional Information:

1. No specimen substitutions.
2. STAT requests are **not accepted** for this test.

3. Submitting less than 1 mL will compromise our ability to perform all necessary testing.

Forms

If not ordering electronically, complete, print, and send a [Therapeutics Test Request](#) (T831) with the specimen.

Specimen Minimum Volume

2.5 mL

Reject Due To

Gross hemolysis	OK
Gross icterus	OK

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	28 days	
	Ambient	14 days	
	Frozen	28 days	

Clinical & Interpretive

Clinical Information

Oxycodone is metabolized to noroxycodone, oxymorphone, and their glucuronides and is excreted primarily via the kidney. The presence of oxycodone indicates exposure to oxycodone within 2 to 3 days prior to specimen collection.

Oxymorphone is metabolized in the liver to noroxymorphone and excreted via the kidney primarily as the glucuronide conjugates. Oxymorphone is also a metabolite of oxycodone and, therefore, the presence of oxymorphone could also indicate exposure to oxycodone.

The detection interval for opiates is generally 2 to 3 days after last ingestion.

Reference Values

Negative

Positive results are reported with a quantitative result.

Cutoff concentrations by liquid chromatography tandem mass spectrometry:

Oxycodone: 25 ng/mL

Noroxycodone: 25 ng/mL

Oxymorphone: 25 ng/mL

Noroxymorphone: 25 ng/mL

**Interpretation**

This procedure reports the total urine concentration; this is the sum of the unconjugated and conjugated forms of the parent drug.

**Cautions**

Other drugs in the opioid class, such as fentanyl, meperidine, methadone, and opiate antagonists such as naloxone, are not detected.

**Clinical Reference**

1. Jutkiewicz EM, Traynor JR. Opioid analgesics. In: Brunton LL, Knollmann BC, eds. Goodman and Gilman's: The Pharmacological Basis of Therapeutics. 14th ed. McGraw-Hill, Inc; 2023:chap 23
2. Baselt, RC. Disposition of Toxic Drugs and Chemical in Man. 10th ed. Biomedical Publications; 2014
3. Hackett LP, Dusci LJ, Ilett KF, Chiswell GM. Optimizing the hydrolysis of codeine and morphine glucuronides in urine. Ther Drug Monit. 2002;24(5):652-657
4. Langman LJ, Bechtel LK, Holstege CP. Clinical toxicology. In: Rifai N, Chiu RWK, Young I, Burnham CAD, Wittwer CT, eds. Tietz Textbook of Laboratory Medicine. 7th ed. Elsevier; 2023:chap 43

**Performance****Method Description**

Confirmation with quantification by liquid chromatography tandem mass spectrometry.(Unpublished Mayo method)

**PDF Report**

No

**Day(s) Performed**

Monday through Friday

**Report Available**

2 to 5 days

**Specimen Retention Time**

14 days

**Performing Laboratory Location**

Mayo Clinic Laboratories - Rochester Superior Drive

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.

# Test Definition: OXYCU

Oxycodone with Metabolite Confirmation,  
Random, Urine

- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

## Test Classification

This test was developed and its performance characteristics determined by Mayo Clinic in a manner consistent with CLIA requirements. It has not been cleared or approved by the US Food and Drug Administration.

## CPT Code Information

G0480  
80365 (if appropriate for select payers)  
[Clinical Toxicology CPT Code Client Guidance](#)

## LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
OXYCU	Oxycodone w/metabolite Conf, U	94304-3

Result ID	Test Result Name	Result LOINC® Value
62616	Oxycodone-by LC-MS/MS	16249-5
35965	Oxymorphone-by LC-MS/MS	17395-5
36022	Oxycodone Interpretation	69050-3
42010	Noroxycodone-by LC-MS/MS	61425-5
42011	Noroxymorphone-by LC-MS/MS	90894-7