

## Overview

### Useful For

Ki67 is a marker of proliferation in neoplasms

Melan A aids in the identification of melanoma

### Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
IHMTO	IHC Multiplex, Tech Only	No	No

### Testing Algorithm

For the initial technical component only immunohistochemical (IHC) stain performed, the appropriate bill-only test ID will be reflexed and charged (IHTOI). For each additional technical component only IHC stain performed, an additional bill-only test ID will be reflexed and charged (IHTOA).

### Method Name

Immunohistochemistry

### NY State Available

Yes

## Specimen

### Specimen Type

TECHONLY

### Ordering Guidance

This test includes only technical performance of the stain (no pathologist interpretation is performed). If diagnostic consultation by a pathologist is required order PATHC / Pathology Consultation.

### Shipping Instructions

Attach the green pathology address label and the pink Immunostain Technical Only label included in the kit to the outside of the transport container.

### Specimen Required

**Supplies:** Immunostain Technical Only Envelope (T693)

**Specimen Type:** Tissue

**Preferred:** 2 Unstained positively charged glass slides (25- x 75- x 1-mm) per test ordered; sections 4-microns thick.

**Acceptable:** Formalin-fixed, paraffin-embedded (FFPE) tissue block

### Digital Image Access

1. Information on accessing digital images of immunohistochemical (IHC) stains and the manual requisition form can be accessed through this website: <https://news.mayocliniclabs.com/ihc-stains/>
2. Clients ordering stains using a manual requisition form will not have access to digital images.
3. Clients wishing to access digital images must place the order for IHC stains electronically. Information regarding digital imaging can be accessed through this website: <https://news.mayocliniclabs.com/ihc-stains/#FAQ>

### Forms

If not ordering electronically, complete, print, and send a [Immunohistochemical \(IHC\)/In Situ Hybridization \(ISH\) Stains Request](#) (T763) with the specimen.

### Reject Due To

Wet/frozen tissue Cytology smears Nonformalin fixed tissue Nonparaffin embedded tissue Noncharged slides ProbeOn slides	Reject
--	--------

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
TECHONLY	Ambient (preferred)		
	Refrigerated		

### Clinical & Interpretive

#### Clinical Information

Ki-67 (clone MIB-1) is a nuclear protein (detected by the chromogen DAB) playing a pivotal role in maintaining cell proliferation. Ki-67 is present in late G1-, S-, M-, and G2-phases of the cell cycle. Cells in the G0 (quiescent) phase are negative for this protein. Melan-A or melanoma antigen recognized by T cells (MART-1) (detected by the chromogen Fast Red) is expressed in the cytoplasm of melanocytes. It is a sensitive and specific marker for the diagnosis of melanoma. Melan A is also found in other tumors of melanocytic origin such as clear cell sarcoma, melanotic neurofibroma, melanotic schwannoma, as well as in perivascular epithelioid cell tumor. Melan A (clone A103) cross-reacts with steroid hormone-producing cells and tumors. Consequently, adrenocortical adenomas/carcinomas and

---

sex cord-stromal tumors of the ovary and testis may exhibit staining.

**Interpretation**

This test includes only technical performance of the stain (no pathologist interpretation is performed). Mayo Clinic cannot provide an interpretation of tech only stains outside the context of a pathology consultation. If an interpretation is needed, refer to PATHC / Pathology Consultation for a full diagnostic evaluation or second opinion of the case. All material associated with the case is required. Additional specific stains may be requested as part of the pathology consultation, and will be performed as necessary at the discretion of the Mayo pathologist.

The positive and negative controls are verified as showing appropriate immunoreactivity and documentation is retained at Mayo Clinic Rochester. If a control tissue is not included on the slide, a scanned image of the relevant quality control tissue is available upon request. Contact 855-516-8404.

Interpretation of this test should be performed in the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

**Cautions**

Age of a cut paraffin section can affect immunoreactivity. Stability thresholds vary widely among published literature and are antigen-dependent. Best practice is for paraffin sections to be cut within 6 weeks.

**Clinical Reference**

1. Puri PK, Valdes CL, Burchette JL, et al: Accurate identification of proliferative index in melanocytic neoplasms with Melan-A/Ki-67 double stain. *J Cutan Pathol* 2010;37(9):1010-1012
2. Nielsen PS, Riber-Hansen R, Steiniche T: Immunohistochemical double stains against Ki67/MART1 and HMB45/MITF: promising diagnostic tools in melanocytic lesions. *Am J Dermatopathol* 2011;33(4):361-370
3. Nielsen PS, Riber-Hansen R, Jensen TO, et al: Proliferation indices of phosphohistone H3 and Ki67: strong prognostic markers in a consecutive cohort with stage I/II melanoma. *Mod Pathol* 2013;26:404-413

**Performance****Method Description**

Immunohistochemistry on sections of paraffin-embedded tissue.(Unpublished Mayo method)

**PDF Report**

No

**Day(s) Performed**

Monday through Friday

**Report Available**

1 to 3 days

**Specimen Retention Time**

---

Until staining is complete.

**Performing Laboratory Location**

Rochester

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**Test Classification**

This test has been cleared, approved, or is exempt by the US Food and Drug Administration and is used per manufacturer's instructions. Performance characteristics were verified by Mayo Clinic in a manner consistent with CLIA requirements.

**CPT Code Information**

88344-TC

**LOINC® Information**

Test ID	Test Order Name	Order LOINC® Value
KIMEL	Ki67+Melan A IHC, Tech Only	Order only;no result

Result ID	Test Result Name	Result LOINC® Value
70795	Ki67+Melan A IHC, Tech Only	Bill only; no result