

Overview

Useful For

Marker of skeletal muscle differentiation

Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
IHTOI	IHC Initial, Tech Only	No	No
IHTOA	IHC Additional, Tech Only	No	No

Testing Algorithm

[For the initial technical component only immunohistochemical \(IHC\) stain performed, the appropriate bill-only test ID will be reflexed and charged \(IHTOI\). For each additional technical component only IHC stain performed, an additional bill-only test ID will be reflexed and charged \(IHTOA\).](#)

Method Name

Immunohistochemistry

NY State Available

Yes

Specimen

Specimen Type

TECHONLY

Ordering Guidance

This test includes only technical performance of the stain (no pathologist interpretation is performed). If diagnostic consultation by a pathologist is required order PATHC / Pathology Consultation.

Shipping Instructions

Attach the green pathology address label and the pink Immunostain Technical Only label included in the kit to the outside of the transport container.

Specimen Required

Supplies: Immunostain Technical Only Envelope (T693)

Specimen Type: Tissue

Preferred: 2 Unstained positively charged glass slide (25- x 75- x 1-mm) per test ordered; sections 4-microns thick.

Acceptable: Formalin-fixed, paraffin-embedded (FFPE) tissue block

Digital Image Access

[Mayo Clinic is preparing to migrate to a new slide scanning solution and will need to temporarily suspend digital imaging for tech-only IHC/ISH orders received after June 17. We will send a follow-up notification by September 1st indicating when digital imaging resumes. We appreciate your patience as we improve our digital imaging solution.](#)

Forms

If not ordering electronically, complete, print, and send a [Immunohistochemical \(IHC\)/In Situ Hybridization \(ISH\) Stains Request](#) (T763) with the specimen.

Reject Due To

Tissue/Other	<ul style="list-style-type: none"> Wet/frozen tissue Cytology smears Nonformalin fixed tissue Nonparaffin embedded tissue Noncharged slides ProbeOn slides
--------------	--

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
TECHONLY	Ambient (preferred)		
	Refrigerated		

Clinical & Interpretive

Clinical Information

Myogenin is a member of a family of myogenic regulatory genes that include *MyoD*, *myf5*, and *MRF4*. These genes encode a set of transcription factors that are essential for muscle development. Expression of myogenin is restricted to cells showing skeletal muscle differentiation. Myogenin is found in the majority of rhabdomyosarcomas and Wilms tumors, and is absent in Ewing sarcoma and mature skeletal muscle.

Interpretation

This test includes only technical performance of the stain (no pathologist interpretation is performed). Mayo Clinic cannot provide an interpretation of tech only stains outside the context of a pathology consultation. If an interpretation is needed, refer to PATHC / Pathology Consultation for a full diagnostic evaluation or second opinion of the case. All material associated with the case is required. Additional specific stains may be requested as part of the pathology consultation, and will be performed as necessary at the discretion of the Mayo pathologist.

The positive and negative controls are verified as showing appropriate immunoreactivity and documentation is retained at Mayo Clinic Rochester. If a control tissue is not included on the slide, a scanned image of the relevant quality control tissue is available upon request. Contact 855-516-8404.

Interpretation of this test should be performed in the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

Cautions

Age of a cut paraffin section can affect immunoreactivity. Stability thresholds vary widely among published literature and are antigen-dependent. Best practice is for paraffin sections to be cut within 6 weeks.

Clinical Reference

1. Cessna MH, Zhou H, Perkins SL, et al: Are myogenin and myoD1 expression specific for rhabdomyosarcoma? Am J Surg Pathol 2001;25(9):1150-1157
2. Folpe AL: MyoD1 and myogenin expression in human neoplasia: A review and update. Adv in Anat Pathol 2002;9(3):198-203
3. Kumar S, Perlman E, Harris CA, et al: Myogenin is a specific marker for rhabdomyosarcoma: an immunohistochemical study in paraffin-embedded tissues. Mod Pathol 2000;13(9):988-993
4. Morgenstern DA, Rees H, Sebire NJ, et al: Rhabdomyosarcoma subtyping by immunohistochemical assessment of myogenin: tissue array study and review of the literature. Pathol and Oncol Res 2008;14(3):233-238

Performance**Method Description**

Immunohistochemistry on sections of paraffin-embedded tissue.(Unpublished Mayo method)

PDF Report

No

Day(s) Performed

Monday through Friday

Report Available

1 to 3 days

Specimen Retention Time

Until staining is complete.

Performing Laboratory Location

Rochester

Fees & Codes**Fees**

-
- Authorized users can sign in to [Test Prices](#) for detailed fee information.
 - Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
 - Prospective clients should contact their Regional Manager. For assistance, contact [Customer Service](#).

Test Classification

This test has been cleared, approved, or is exempt by the US Food and Drug Administration and is used per manufacturer's instructions. Performance characteristics were verified by Mayo Clinic in a manner consistent with CLIA requirements.

CPT Code Information

88342-TC, primary

88341-TC, if additional IHC

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
MYOGE	Myogenin IHC, Tech Only	Order only;no result

Result ID	Test Result Name	Result LOINC® Value
70828	Myogenin IHC, Tech Only	Bill only; no result