

## Overview

### Useful For

Classification of neurodegenerative diseases

### Testing Algorithm

[For the initial technical component only immunohistochemical \(IHC\) stain performed, the appropriate bill-only test ID will be reflexed and charged \(IHTOI\). For each additional technical component only IHC stain performed, an additional bill-only test ID will be reflexed and charged \(IHTOA\).](#)

### Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
IHTOA	IHC Additional, Tech Only	No	No
IHTOI	IHC Initial, Tech Only	No	No

### Method Name

Immunohistochemistry

### NY State Available

Yes

## Specimen

### Specimen Type

TECHONLY

### Ordering Guidance

This test includes only technical performance of the stain (no pathologist interpretation is performed). If diagnostic consultation by a pathologist is required order PATHC / Pathology Consultation.

### Shipping Instructions

Attach the green pathology address label and the pink Immunostain Technical Only label included in the kit to the outside of the transport container.

### Specimen Required

**Supplies:** Immunostain Technical Only Envelope (T693)

**Specimen Type:** Tissue

**Preferred:** 2 Unstained positively charged glass slide (25- x 75- x 1-mm) per test ordered; sections 4-microns thick.

**Acceptable:** Formalin-fixed, paraffin-embedded (FFPE) tissue block

### Digital Image Access

1. Information on accessing digital images of IHC stains and the manual requisition form can be accessed through this website: <https://news.mayocliniclabs.com/ihc-stains/>
2. Clients ordering stains using a manual requisition form will not have access to digital images.
3. Clients wishing to access digital images must place the order for IHC stains electronically. Information regarding digital imaging can be accessed through this website: <https://news.mayocliniclabs.com/ihc-stains/#FAQ>

### Forms

If not ordering electronically, complete, print, and send a [Immunohistochemical \(IHC\)/In Situ Hybridization \(ISH\) Stains Request](#) (T763) with the specimen.

### Reject Due To

Tissue/Slides    Wet/frozen tissue  
                          Cytology smears  
                          Nonformalin fixed tissue  
                          Nonparaffin embedded tissue  
                          Noncharged slides  
                          ProbeOn slides

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
TECHONLY	Ambient (preferred)		
	Refrigerated		

### Clinical & Interpretive

#### Clinical Information

Ubiquitin is a polypeptide of approximately 8.5 kD found in filamentous inclusions and cytosome-related organelles in human idiopathic neurodegenerative diseases, including Alzheimer disease, Pick disease, Lewy body dementia, and Parkinson disease. Ubiquitin is also expressed in Rosenthal fibers in astrocytomas. Ubiquitin protein complexes have also been found in primary lysosome-related granules in mature neutrophils. Ubiquitin labels the periphery of senile plaques and of neurofibrillary tangles in Alzheimer disease, Lewy bodies in Parkinson disease, and Mallory bodies in alcoholic liver disease.

---

**Interpretation**

This test includes only technical performance of the stain (no pathologist interpretation is performed). Mayo Clinic cannot provide an interpretation of tech only stains outside the context of a pathology consultation. If an interpretation is needed, refer to PATHC / Pathology Consultation for a full diagnostic evaluation or second opinion of the case. All material associated with the case is required. Additional specific stains may be requested as part of the pathology consultation, and will be performed as necessary at the discretion of the Mayo pathologist.

The positive and negative controls are verified as showing appropriate immunoreactivity and documentation is retained at Mayo Clinic Rochester. If a control tissue is not included on the slide, a scanned image of the relevant quality control tissue is available upon request. Contact 855-516-8404.

Interpretation of this test should be performed in the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

**Cautions**

Age of a cut paraffin section can affect immunoreactivity. Stability thresholds vary widely among published literature and are antigen-dependent. Best practice is for paraffin sections to be cut within 6 weeks.

**Clinical Reference**

- [1. Chu CT, Caruso JL, Cummings TJ, et al: Ubiquitin immunochemistry as a diagnostic aid for community pathologists evaluating patients who have dementia. Mod Pathol 2000;13\(4\):420-426](#)
2. Josephs KA, Holton JL, Rossor MN, et al: Frontotemporal lobar degeneration and ubiquitin immunohistochemistry. Neuropathol Applied Neurobiol 2004;30:369-373
3. Katsuse O, Dickson DW: Ubiquitin immunohistochemistry of frontotemporal lobar degeneration differentiates cases with and without motor neuron disease. Alzheimer Dis Assoc Disord 2005;19:S37-S43
4. Lennox G, Lowe J, Landon M, et al: Diffuse Lewy body disease: correlative neuropathology using anti-ubiquitin immunocytochemistry. J Neurol, Neurosurg Psychiatry 1989;52:1236-1247
5. Mackenzie IRA, Feldman HH: Ubiquitin immunohistochemistry suggests classic motor neuron disease, motor neuron disease with dementia, and frontotemporal dementia of the motor neuron disease type represent a clinicopathologic spectrum. J Neuropathol Exp Neurol 2005;64(8):730-739

**Performance****Method Description**

---

Immunohistochemistry on sections of paraffin-embedded tissue.(Unpublished Mayo method)

**PDF Report**

No

**Specimen Retention Time**

Until staining is complete.

**Performing Laboratory Location**

Rochester

**Fees & Codes****Test Classification**

This test has been cleared, approved, or is exempt by the US Food and Drug Administration and is used per manufacturer's instructions. Performance characteristics were verified by Mayo Clinic in a manner consistent with CLIA requirements.

**CPT Code Information**

88342-TC, primary

88341-TC, if additional IHC

**LOINC® Information**

Test ID	Test Order Name	Order LOINC Value
UBIQ	Ubiquitin IHC, Tech Only	Order only;no result

Result ID	Reporting Name	LOINC®
70890	Ubiquitin IHC, Tech Only	Bill only; no result