

Overview

Useful For

Identification of the presence of achaete-scute homolog 1 (ASCL1)

Testing Algorithm

For the initial technical component only immunohistochemical (IHC) stain performed, the appropriate bill-only test ID will be reflexed and charged (IHTOI). For each additional technical component only IHC stain performed, an additional bill-only test ID will be reflexed and charged (IHTOA).

Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
IHTOI	IHC Initial, Tech Only	No	No
IHTOA	IHC Additional, Tech Only	No	No

Method Name

Immunohistochemistry (IHC)

NY State Available

Yes

Specimen

Specimen Type

TECHONLY

Ordering Guidance

This test includes only technical performance of the stain (no pathologist interpretation is performed). If diagnostic consultation by a pathologist is required order PATHC / Pathology Consultation.

Shipping Instructions

Attach the green pathology address label and the pink Immunostain Technical Only label included in the kit to the outside of the transport container.

Specimen Required

Supplies: Immunostain Technical Only Envelope (T693)

Specimen Type: Tissue

Container/Tube: [Immunostain Technical Only Envelope](#)

Preferred: 2 Unstained positively charged glass slide (25- x 75- x 1-mm) per test ordered; sections 4-microns thick.

Acceptable: Formalin-fixed, paraffin-embedded (FFPE) tissue block

Digital Image Access

1. Information on accessing digital images of immunohistochemical (IHC) stains and the manual requisition form can be accessed through this website: www.mayocliniclabs.com/test-info/ihc/index.html

2. Clients ordering stains using a manual requisition form will not have access to digital images.
3. Clients wishing to access digital images must place the order for IHC stains electronically. Information regarding digital imaging can be accessed through this website: www.mayocliniclabs.com/test-info/ihc/faq.html

Forms

If not ordering electronically, complete, print, and send a [Immunohistochemical \(IHC\)/In Situ Hybridization \(ISH\) Stains Request](#) (T763) with the specimen.

Reject Due To

Wet/frozen tissue Reject
 Cytology smears
 Nonformalin fixed tissue
 Nonparaffin embedded tissue
 Noncharged slides
 ProbeOn slides

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
TECHONLY	Ambient (preferred)		
	Refrigerated		

Clinical & Interpretive

Clinical Information

Achaete-scute homolog 1 (ACSL1), alternatively titled hASH1 or MASH1, is a member of the basic helix-loop-helix family of transcription factors. ACSL1 may play a role at early stages of development of specific neural lineages in most regions of the central nervous system, and of several lineages in the peripheral nervous system. The protein has been shown to be highly expressed in medullary thyroid cancer and small cell lung cancer and may be a useful marker for these cancers.

Interpretation

This test does not include pathologist interpretation only technical performance of the stain. If interpretation is required order PATHC / Pathology Consultation for a full diagnostic evaluation or second opinion of the case.

The positive and negative controls are verified as showing appropriate immunoreactivity and documentation is retained at Mayo Clinic Rochester. If a control tissue is not included on the slide, a scanned image of the relevant quality control tissue is available upon request, call 855-516-8404.

Interpretation of this test should be performed in the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

Cautions

Age of a cut paraffin section can affect immunoreactivity. Stability thresholds vary widely among published literature and are antigen dependent. Best practice is for paraffin sections to be cut within 6 weeks.

Clinical Reference

1. Au NH, Cheang M, Huntsman DG, et al: Evaluation of immunohistochemical markers in non-small cell lung cancer by unsupervised hierarchical clustering analysis: a tissue microarray study of 284 cases and 18 markers. J Pathol.

2004;204:101-109

2. Kosari F, Ida CM, Aubry M-C, et al: ASCL1 and RET expression defines a clinically relevant subgroup of lung adenocarcinoma characterized by neuroendocrine differentiation. *Oncogene*. 2014;33:3776-3783

3. Ralston J, Chiriboga L, Nonaka D: MASH1: a useful marker in differentiating pulmonary small cell carcinoma from merkel cell carcinoma. *Mod Pathol*. 2008 Nov;21(11):1357-1362

4. Somasundaram K, Reddy SP, Vinnakota K, et al: Upregulation of ASCL1 and inhibition of Notch signaling pathway characterize progressive astrocytoma. *Oncogene*. 2005;24:7073-7083

Performance

Method Description

Immunohistochemistry on sections of paraffin-embedded tissue.(Cartun RW, Taylor CR, Dabbs DJ: Techniques of immunohistochemistry: Principles, pitfalls, and standardization. In: Dabbs DJ, ed. *Diagnostic Immunohistochemistry*. 5th ed. Elsevier; 2019:1-46)

PDF Report

No

Specimen Retention Time

Until staining is complete.

Performing Laboratory Location

Rochester

Fees & Codes

Test Classification

This test has been cleared, approved, or is exempt by the US Food and Drug Administration and is used per manufacturer's instructions. Performance characteristics were verified by Mayo Clinic in a manner consistent with CLIA requirements.

CPT Code Information

88342-TC, primary

88341-TC, if additional IHC