

## Overview

### Useful For

Identifying the presence of *CALR* exon 9 frameshift alterations in myeloproliferative neoplasms

### Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
IHTOI	IHC Initial, Tech Only	No	No
IHTOA	IHC Additional, Tech Only	No	No

### Testing Algorithm

[For the initial technical component only immunohistochemical \(IHC\) stain performed, the appropriate bill-only test ID will be reflexed and charged \(IHTOI\). For each additional technical component only IHC stain performed, an additional bill-only test ID will be reflexed and charged \(IHTOA\).](#)

### Method Name

Immunohistochemistry

### NY State Available

Yes

## Specimen

### Specimen Type

TECHONLY

### Ordering Guidance

This test includes only technical performance of the stain (no pathologist interpretation is performed). If diagnostic consultation by a pathologist is required order PATHC / Pathology Consultation.

### Shipping Instructions

Attach the green pathology address label and the pink Immunostain Technical Only label included in the kit to the outside of the transport container.

### Specimen Required

**Supplies:** Immunostain Technical Only Envelope (T693)

**Specimen Type:** Tissue

**Container/Tube:** Immunostain Technical Only Envelope

**Preferred:** 2 Unstained positively charged glass slide (25- x 75- x 1-mm) per test ordered; sections 4-microns thick

**Acceptable:** Formalin-fixed, paraffin-embedded (FFPE) tissue block

### Digital Image Access

1. Information on accessing digital images of immunohistochemical (IHC) stains and the manual requisition form can be accessed through this website: <https://news.mayocliniclabs.com/ihc-stains/>
2. Clients ordering stains using a manual requisition form will not have access to digital images.
3. Clients wishing to access digital images must place the order for IHC stains electronically. Information regarding digital imaging can be accessed through this website: <https://news.mayocliniclabs.com/ihc-stains/#FAQ>

### Forms

If not ordering electronically, complete, print, and send a [Immunohistochemical \(IHC\)/In Situ Hybridization \(ISH\) Stains Request](#) (T763) with the specimen.

### Reject Due To

Wet/frozen tissue Cytology smears Nonformalin fixed tissue Nonparaffin embedded tissue Noncharged slides ProbeOn slides	Reject
--	--------

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
TECHONLY	Ambient (preferred)		
	Refrigerated		

### Clinical & Interpretive

#### Clinical Information

The detection of calreticulin exon 9 frameshift alterations can assist in the diagnosis and prognostication of myeloproliferative diseases. Although these alterations are heterogeneous, they all result in a protein with a novel 36-amino acid C terminus the anticalreticulin CAL2 clone specifically identifies.

Most patients with essential thrombocythemia or primary myelofibrosis not associated with Janus kinase 2 (*JAK2*) or *MPL* variants are associated with *CALR* exon 9 variants and primary myelofibrosis patients carrying *CALR* variants have a more indolent clinical course.

**Interpretation**

This test does not include pathologist interpretation; if an interpretation is required order PATHC / Pathology Consultation for a full diagnostic evaluation or second opinion of the case.

The positive and negative controls are verified as showing appropriate immunoreactivity and documentation is retained at Mayo Clinic Rochester. If a control tissue is not included on the slide, a scanned image of the relevant quality control tissue is available upon request. Contact 855-516-8404.

Interpretation of this test should be performed in the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

**Cautions**

Age of a cut paraffin section can affect immunoreactivity. Stability thresholds vary widely among published literature and are antigen-dependent. Best practice is for paraffin sections to be cut within 6 weeks.

**Clinical Reference**

1. Vannucchi AM, Rotunno G, Bartalucci N, et al: Calreticulin mutation-specific immunostaining in myeloproliferative neoplasms: pathogenetic insight and diagnostic value. *Leukemia*. 2014 Sep;28(9):1811-1818. doi: 10.1038/leu.2014.100
2. Stein H, Bob R, Durkop H, et al: A new monoclonal antibody (CAL2) detects CALRETICULIN mutations in formalin-fixed and paraffin-embedded bone marrow biopsies. *Leukemia*. 2016 Jan;30(1):131-135. doi: 10.1038/leu.2015.192
3. Klampfl T, Gisslinger H, Harutyunyan AS, et al: Somatic mutations of calreticulin in myeloproliferative neoplasms. *N Engl J Med*. 2013 Dec 19;369(25):2379-2390. doi: 10.1056/NEJMoa1311347

**Performance****Method Description**

Immunohistochemistry on sections of paraffin-embedded tissue.(Unpublished Mayo method)

**PDF Report**

No

**Day(s) Performed**

Monday through Friday

**Report Available**

1 to 3 days

**Specimen Retention Time**

Until staining is complete.

**Performing Laboratory Location**

Rochester

---

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**Test Classification**

This test has been cleared, approved, or is exempt by the US Food and Drug Administration and is used per manufacturer's instructions. Performance characteristics were verified by Mayo Clinic in a manner consistent with CLIA requirements.

**CPT Code Information**

88342-TC, primary

88341-TC, if additional IHC

**LOINC® Information**

Test ID	Test Order Name	Order LOINC® Value
CALRC	Calreticulin ex9mut IHC, Tech Only	Order only;no result

Result ID	Test Result Name	Result LOINC® Value
71493	Calreticulin ex9mut IHC, Tech Only	Bill only; no result