

Overview

Testing Algorithm

If the Amphetamine, Scr w-Conf, S is positive, then the Amphetamines, Rfx (FASCR) will be performed at an additional charge.

Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
FASCR	Amphetamines, Rfx	No	No

Method Name

Immunoassay (IA) â€œ Screen

Gas Chromatography/Mass Spectrometry (GS/MS) â€œ Confirmation; if applicable

NY State Available

Yes

Specimen

Specimen Type

Serum Red

Specimen Required

Draw blood in a plain red-top tube(s), **serum gel tube is not acceptable**. Spin down and send 7 mL of serum refrigerated in a plastic vial.

Reject Due To

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	NA

Specimen Minimum Volume

3 mL

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	14 days	
	Frozen	180 days	
	Ambient		

Clinical & Interpretive

Reference Values

Reference Range:

Amphetamines: Cutoff: 50

Confirmation Threshold: 10 mg/mL

Performance

PDF Report

No

Performing Laboratory Location

Medtox Laboratories, Inc.

Fees & Codes

Test Classification

CPT Code Information

80307

80324, 80359-if applicable

LOINC® Information

Test ID	Test Order Name	Order LOINC Value
FASCC	Amphetamines, Scr w-Conf, S	8149-7

Result ID	Reporting Name	LOINC®
Z4631	Amphetamines	8149-7