

Overview

Testing Algorithm

If appropriate, confirmation testing will be performed at an additional charge.

Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
FAMCS	Amphetamines Confirmation, S	No	No
FBBCS	Barbiturates Confirmation, S	No	No
FBCS	Benzodiazepines Confirmation, S	No	No
FCANS	Cannabinoids Confirmation, S	No	No
FCCM	Cocaine and Metabolites Confirm, S	No	No
FMMCS	Methadone and Metabolite Confirm, S	No	No
FOFU	Opiates-Free Unconjugated Conf, S	No	No
FPCCS	Phencyclidine Confirmation, S	No	No

Method Name

Enzyme-Linked Immunosorbent Assay (ELISA)

NY State Available

Yes

Specimen

Specimen Type

Serum Red

Specimen Required

Specimen Type: Serum

Container/Tube: Red top

Specimen Volume: 4 mL

Collection Instructions: Draw blood in a plain, red-top tube(s), serum gel tube is not acceptable. Spin down and send 4 mL of serum refrigerated in plastic, preservative-free vial.

Reject Due To

Hemolysis NA
 Lipemia NA
 Icterus NA
 Other Polymer gel separation tube (SST or PST).

Specimen Minimum Volume

2 mL

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	7 days	
	Frozen	14 days	

Clinical & Interpretive

Reference Values

Reporting limit determined each analysis.

<u>Test</u>	<u>Result</u>	<u>Units</u>
Opiates	None Detected	ng/mL
Cocaine/Metabolites	None Detected	ng/mL
Benzodiazepines	None Detected	ng/mL
Cannabinoids	None Detected	ng/mL
Amphetamines	None Detected	ng/mL
Barbiturates	None Detected	ng/mL
Methadone/Metabolite	None Detected	ng/mL
Phencyclidine	None Detected	ng/mL
Methamphetamine/MDMA	None Detected	ng/mL
Oxycodone/Oxymorphone	None Detected	ng/mL

Performance

PDF Report

No

Performing Laboratory Location

NMS Labs

Fees & Codes**Test Classification**

This test was developed and its performance characteristics determined by NMS Labs. It has not been cleared or approved by the U.S. Food and Drug Administration.

CPT Code Information

80307

LOINC® Information

Test ID	Test Order Name	Order LOINC Value
FD10S	Drugs of Abuse Screen (10 Panel), S	Not Provided

Result ID	Reporting Name	LOINC®
Z5245	Opiates	8219-8
Z5246	Cocaine/Metabolites	73823-7
Z5247	Benzodiazepines	46976-7
Z5248	Cannabinoids	8172-9
Z5249	Amphetamines	8149-7
Z5250	Barbiturates	20421-4
Z5251	Methadone/Metabolite	59705-4
Z5252	Phencyclidine	8236-2
Z5253	Methamphetamine/MDMA	91034-9
Z5254	Oxycodone/Oxymorphone	77732-6