

Overview**Method Name**

High Performance Liquid Chromatography/Tandem Mass Spectrometry (LC-MS/MS)

NY State Available

Yes

Specimen**Specimen Type**

Serum Red

Specimen Required

Draw blood in a plain, red-top tube(s). (Serum gel tube is not acceptable.) Spin down and send 1 mL of serum refrigerate in a preservative-free plastic vial.

Specimen Minimum Volume

0.5 mL

Reject Due To

Other	Serum gel tube (SST)
-------	----------------------

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	14 days	
	Frozen	120 days	
	Ambient	14 days	

Clinical & Interpretive**Reference Values**

Reporting limit determined each analysis.

Units: mcg/mL

Interpretation

Serum concentrations required for therapeutic effects reportedly range from 0.08 - 0.40 mcg/mL.

Performance**PDF Report**

No

Day(s) Performed

Monday - Sunday

Report Available

9 to 11 days

Performing Laboratory Location

NMS Labs

Fees & Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by NMS Labs. It has not been cleared or approved by the U.S. Food and Drug Administration.

CPT Code Information

80369

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FBACS	Baclofen, Serum	9353-4

Result ID	Test Result Name	Result LOINC® Value
Z3532	Baclofen	9353-4