
Overview**Method Name**

Liquid Chromatography/Tandem Mass Spectrometry (LC/MS/MS)

NY State Available

Yes

Specimen**Specimen Type**

Varies

Specimen Required**Submit only 1 of the following specimens:****Plasma**

Draw blood in a green-top (sodium heparin) tube(s), **plasma gel tube is not acceptable**. Spin down and send 2 mL sodium heparin plasma frozen in plastic vial.

Serum

Draw blood in a plain, red-top tube(s), **serum gel tube is not acceptable**. Spin down and send 2 mL of serum frozen in plastic vial.

Reject Due To

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	NA

Specimen Minimum Volume

0.3 mL

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Varies	Frozen (preferred)	180 days	

Clinical & Interpretive**Reference Values**

Flurazepam:

Reference Range: 0 - 30 ng/mL

Desalkylflurazepam:

Reference Range: 30 - 150 ng/mL

Flurazepam + Desalkylflurazepam:

Reference Range: 30 - 180 ng/mL

Performance**PDF Report**

No

Specimen Retention Time

2 weeks

Performing Laboratory Location

Medtox Laboratories, Inc.

Fees & Codes

Test Classification**CPT Code Information**

80346

LOINC® Information

Test ID	Test Order Name	Order LOINC Value
FFLUR	Flurazepam (Dalmane)	73826-0

Result ID	Reporting Name	LOINC®
Z1133	Flurazepam	46979-1
Z1160	Desalkylflurazepam	46980-9
Z1161	Flurazepam + Desalkylflurazepam	73826-0