
Overview**Method Name**

High Performance Liquid Chromatography with Fluorescence

NY State Available

Yes

Specimen**Specimen Type**

Varies

Specimen Required**Submit only 1 of the following specimens:****Plasma**

Draw blood in a green-top (sodium heparin) tube(s), **plasma gel tube is not acceptable**. Spin down and send 1 mL sodium heparin plasma refrigerated in a plastic vial.

Serum

Draw blood in a plain red-top tube(s), **serum gel tube is not acceptable**. Spin down and send 1 mL of serum refrigerated in a plastic vial.

Reject Due To

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	NA

Specimen Minimum Volume

0.25 mL

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	7 days	
	Frozen	180 days	
	Ambient		

Clinical & Interpretive**Reference Values**

Expected serum furosemide concentration in patients on usual daily dosages: Up to 5.0 ug/mL

Toxic: greater than 50.0 ug/mL

Performance**PDF Report**

No

Specimen Retention Time

2 weeks

Performing Laboratory Location

Medtox Laboratories, Inc.

Fees & Codes**Test Classification****CPT Code Information**

80299

LOINC® Information

Test ID	Test Order Name	Order LOINC Value
FFURO	Furosemide (Lasix)	3659-0

Result ID	Reporting Name	LOINC®
Z1189	Furosemide	3659-0