

---

**Overview****Method Name**

Enzymatic, Colorimetric Assay

**NY State Available**

Yes

**Specimen****Specimen Type**

Varies

**Specimen Required**

Submit only 1 of the following specimens:

**Serum**

Draw blood in a plain red-top tube(s), serum gel tube is acceptable. Spin down and send 1 mL of serum ambient in a plastic vial.

**Plasma**

Draw blood in a lavender-top (EDTA) tube(s). Spin down and send 1 mL of EDTA plasma ambient in a plastic vial.

**Reject Due To**

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	NA

**Specimen Minimum Volume**

0.75 mL Note: This volume does not allow for repeat testing.

**Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Varies	Ambient (preferred)	7 days	
	Refrigerated	7 days	
	Frozen		

## Clinical & Interpretive

### Reference Values

Age                      Range

<18 y                    Not Established

Adult Males            10.7-32.0

Adult Females        6.8-29.3

Glycemic control goal for diabetic patients: >10

GlycoMark is intended for use with managing glycemic control in diabetic patients. A low result corresponds to high glucose peaks.

1, 5-AG blood levels can be affected by clinical conditions or medications.

## Performance

### PDF Report

No

### Performing Laboratory Location

Esoterix Endocrinology

## Fees & Codes

### Test Classification

### CPT Code Information

---

84378

**LOINC® Information**

Test ID	Test Order Name	Order LOINC Value
FGLMA	GlycoMark	53835-5

Result ID	Reporting Name	LOINC®
Z0275	GlycoMark	53835-5