

## Overview

### Method Name

Gas Chromatography-Mass Spectrometry (GC/MS)

### NY State Available

Yes

## Specimen

### Specimen Type

Varies

### Specimen Required

Submit only 1 of the following specimens:

#### Plasma

Draw blood in a green-top (sodium heparin) tube(s), **plasma gel tube is not acceptable**. Spin down and send 5 mL sodium heparin plasma refrigerated in a plastic vial.

#### Serum

Draw blood in a plain red-top tube(s), **serum gel tube is not acceptable**. Spin down and send 5 mL of serum refrigerated in a plastic vial.

### Reject Due To

Hemolysis:	NA
Thawing:	Warm OK: Cold OK
Lipemia:	NA
Icterus:	NA
Other	NA

### Specimen Minimum Volume

2 mL

**Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	14 days	
	Frozen	365 days	
	Ambient		

**Clinical & Interpretive****Reference Values**

Units: Flunitrazepam      ng/mL

7-Aminoflunitrazepam    ng/mL

Peak plasma Flunitrazepam concentrations in patients receiving chronic, recommended dosages: 10 – 20 ng/mL.

Note: Flunitrazepam is not legally marketed in the United States.

**Performance****PDF Report**

No

**Performing Laboratory Location**

Medtox Laboratories, Inc.

**Fees & Codes****Test Classification****CPT Code Information**

80346

---

**LOINC® Information**

Test ID	Test Order Name	Order LOINC Value
FFLRO	Flunitrazepam (Rohypnol)	Not Provided

Result ID	Reporting Name	LOINC®
Z1075	Flunitrazepam	73828-6
Z1074	7-Aminoflunitrazepam	73829-4