
Reporting Title: HCV Ab Cadaver/Hemolyzed, S**Performing Location** Rochester**Ordering Guidance:**

For testing hemolyzed specimens from **asymptomatic** patients with or without risk factors for hepatitis C virus (HCV) infection, order HCCAD / Hepatitis C Virus Antibody Screen for Cadaveric or Hemolyzed Specimens, Asymptomatic, Serum.

Necessary Information:**Date of collection is required.****Specimen Requirements:****Collection Container/Tube:****Preferred:** Serum gel**Acceptable:** Red top**Submission Container/Tube:** Plastic vial**Specimen Volume:** 0.5 mL**Collection Instructions:**

1. Centrifuge blood collection tube per collection tube manufacturer's instructions (eg, centrifuge within 2 hours of collection for BD Vacutainer tubes).
2. Aliquot serum into plastic vial.

Forms:

If not ordering electronically, complete, print, and send a [Gastroenterology and Hepatology Client Test Request](#) (T728) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum	Frozen (preferred)	28 days	
	Ambient	7 days	
	Refrigerated	7 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC
58127	HCV Ab Cadaver/Hemolyzed, S	Alphanumeric		13955-0

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

86803

86804 (if appropriate)

Reflex Tests:

Test Id	Reporting Name	CPT Units	CPT Code	Always Performed	Available Separately
HCVL	HCV Ab Confirmation, S	1	86804	No	Yes

Result Codes for Reflex Tests:

Test ID	Result ID	Reporting Name	Type	Unit	LOINC
HCVL	63063	HCV Ab Confirmation, S	Alphanumeric		40726-2

Reference Values:

Negative