
Reporting Title: HBs Ab Monitor, Post-transplant, S**Performing Location** Rochester**Ordering Guidance:**

This test is not useful for determining past hepatitis B or immune status after hepatitis B virus (HBV) vaccination and it does not provide interpretation of the anti-hepatitis B surface (anti-HBs) level detected; order HBAB / Hepatitis B Surface Antibody, Qualitative/Quantitative, Serum for those situations.

Necessary Information:**Date of collection is required.****Specimen Requirements:****Collection Container/Tube:** Serum gel**Submission Container/Tube:** Plastic vial**Specimen Volume:** 1 mL**Collection Instructions:**

1. Centrifuge blood collection tube per collection tube manufacturer's instructions.
2. Aliquot serum into plastic vial.

Forms:

If not ordering electronically, complete, print, and send a [Gastroenterology and Hepatology Client Test Request](#) (T728) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum SST	Frozen (preferred)	28 days	
	Refrigerated	7 days	
	Ambient	24 hours	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC
HBABT	HBs Ab Monitor, Post-transplant, S	Alphanumeric	mIU/mL	5193-8

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

86317

Reference Values:

Not applicable