

---

**Reporting Title:** Cocaine and metabolite Conf, U**Performing Location** Rochester**Ordering Guidance:**

1. For situations where chain of custody is required, a Chain-of-Custody Kit (T282) is available. For chain-of-custody testing, order COKEX / Cocaine and Metabolite Confirmation, Chain of Custody, Urine.
2. Additional drug panels and specific requests are available. Call 800-533-1710 or 507-266-5700.
3. If urine creatinine is required or adulteration of the sample is suspected, order ADULT / Adulterants Survey, Urine.

**Specimen Requirements:****Supplies:** Urine Tubes, 10 mL (T068)**Collection Container/Tube:** Plastic urine container**Submission Container/Tube:** 10-mL urine tube**Specimen Volume:** 10 mL**Collection Instructions:**

1. Collect a random urine specimen.
2. No preservative.

**Additional Information:**

1. No specimen substitutions.
2. STATS are **not** accepted for this test.

**Forms:**If not ordering electronically, complete, print, and send a [Therapeutics Test Request](#) (T831) with the specimen.

| Specimen Type | Temperature              | Time     | Special Container |
|---------------|--------------------------|----------|-------------------|
| Urine         | Refrigerated (preferred) | 14 days  |                   |
|               | Frozen                   | 14 days  |                   |
|               | Ambient                  | 72 hours |                   |

**Supplemental Report:**

No

**CPT Code Information:**

80353

G0480 (if appropriate)

**Reference Values:**

Negative

Positive results are reported with a quantitative GC-MS result.

Cutoff concentrations:

COCAINE BY GC-MS

&lt;50 ng/mL

BENZOYLECGONINE BY GC-MS

&lt;50 ng/mL