

Reporting Title: UroVysion (R) for Bladder Cancer
Performing Location: Rochester

Necessary Information:

- 1. **Specimen source is required** on request form (ie, voided, catheterized, bladder washing).
- 2. Provide fixative, source, reason for referral (evaluate for urothelial carcinoma or hematuria) and status of diagnosis (known previous diagnosis or suspected/unknown).

Specimen Requirements:

Specimen Type: Urine

Sources: Voided urine, catheterized urine, bladder washings, stoma collections, ureteral brushings or washings, renal pelvic brushings or washings

Supplies: FISH for Urothelial Carcinoma Urine Collection Kit (T509)

Container/Tube:

Preferred: FISH for Urothelial Carcinoma in Urocyte Urine Collection Kit

Acceptable: 70% ethanol, PreservCyt, CytoLyt, ThinPrep UroCyt (UroCyt PreservCyt Solution)

Specimen Volume: 30 mL

Collection Instructions:

- 1. Follow instructions included with Urocyte Urine Collection Kit.
- 2. If kit is not used, submit a random urine specimen with an equal volume of 70% ethanol, PreservCyt, or CytoLyt.

Forms:

- 1. [Pathology/Cytology Information](#) (T707)
- 2. If not ordering electronically, complete, print, and send an [Oncology Test Request](#) (T729) with the specimen.

Specimen Type	Temperature	Time	Special Container
Varies	Ambient		
	Refrigerated (preferred)		

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
54674	Result Summary	Alphanumeric		50397-9
54675	Result	Alphanumeric		82251-0
54676	Interpretation	Alphanumeric		69047-9
54677	Reason for Referral	Alphanumeric		42349-1
54678	Specimen	Alphanumeric		31208-2
54679	Source	Alphanumeric		31208-2
54680	Released By	Alphanumeric		18771-6

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

88120

Reference Values:

An interpretive report will be provided.