

Test Definition: KS

Potassium, Serum

Reporting Title: Potassium, S

Performing Location: Mayo Clinic Laboratories - Rochester Main Campus

Necessary Information:

Patient's age and sex are required.

Specimen Requirements:

Collection Container/Tube:

Preferred: Serum gel **Acceptable:** Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 0.5 mL **Collection Instructions:**

- 1. Serum gel tubes should be centrifuged within 2 hours of collection.
- 2. Red-top tubes should be centrifuged, and the serum aliquoted into a plastic vial within 2 hours of collection.

Forms:

If not ordering electronically, complete, print, and send 1 of the following forms with the specimen:

- -Kidney Transplant Test Request
- -Renal Diagnostics Test Request (T830)

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated	48 hours	

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
KS	Potassium, S	Numeric	mmol/L	2823-3

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

84132

Reference Values:

<1 year: not established > or =1 year: 3.6-5.2 mmol/L