

**Reporting Title:** Perampanel, S**Performing Location:** Mayo Clinic Laboratories - Rochester Superior Drive**Specimen Requirements:****Supplies:** Sarstedt Aliquot Tube, 5 mL (T914)**Collection Container/Tube:**Preferred: Red top (serum gel/SST is **not acceptable**)

Acceptable: None

**Submission Container/Tube:** Plastic vial**Specimen Volume:** 1 mL**Collection Instructions:**

1. Draw blood immediately before next scheduled dose.
2. For sustained-release formulations ONLY, draw blood a minimum of 12 hours after last dose.
3. Within 2 hours of collection, centrifuge and aliquot serum into a plastic vial.

**Forms:**

If not ordering electronically, complete, print, and send 1 of the following forms with the specimen:

[-Therapeutics Test Request](#) (T831)[-Neurology Specialty Testing Client Test Request](#) (T732)

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	28 days	
	Ambient	28 days	
	Frozen	28 days	

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
609438	Perampanel, S	Numeric	ng/mL	88895-8

LOINC® and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

**CPT Code Information:**

80299

**Reference Values:**

180-980 ng/mL