



Test Definition: RPRT3

Rapid Plasma Reagin (RPR) with Reflex to
Treponema pallidum Particle Agglutination,
Serum

Reporting Title: RPR w/ Reflex to TP-PA, S
Performing Location: Mayo Clinic Jacksonville Clinical Lab

Ordering Guidance:

This assay should only be used following an initial reactive treponemal antibody assay result as part of the reverse syphilis testing algorithm.

Specimen Requirements:

Supplies: Sarstedt Aliquot Tube, 5 mL (T914)

Collection Container/Tube:

Preferred: Serum gel

Acceptable: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 0.3 mL

Collection Instructions: Centrifuge and aliquot serum into a plastic vial.

Forms:

If not ordering electronically, complete, print, and send an [Infectious Disease Serology Test Request](#) (T916) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	14 days	
	Frozen	14 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
616970	RPR w/ Reflex to TP-PA, S	Alphanumeric		20507-0

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

86592

86593- (if appropriate)

86780- (if appropriate)

Reflex Tests:

Test Id	Reporting Name	CPT Units	CPT Code	Always Performed	Available Separately
---------	----------------	-----------	----------	------------------	----------------------

Test Definition: RPRT3

Rapid Plasma Reagin (RPR) with Reflex to
Treponema pallidum Particle Agglutination,
Serum

RPRT4	RPR, Titer, S	1	86593	No	No
RTPPA	Syphilis Ab, TP-PA, S	1	86780	No	Yes, (Order TPPA)

Result Codes for Reflex Tests:

Test ID	Result ID	Reporting Name	Type	Unit	LOINC®
RTPPA	34512	Syphilis Ab, TP-PA, S	Alphanumeric		24312-1
RPRT4	616971	RPR, Titer, S	Alphanumeric		31147-2

Reference Values:

Negative

Reference values apply to all ages.