

Ferritin, Serum

## Reporting Title: Ferritin, S

Performing Location: Mayo Clinic Laboratories - Rochester Main Campus

## **Specimen Requirements:**

Patient Preparation: For 12 hours before specimen collection, patient should not take multivitamins or dietary supplements (eg, hair, skin, and nail supplements) containing biotin (vitamin B7).
Collection Container/Tube:
Preferred: Serum gel
Acceptable: Red top
Submission Container Tube: Plastic vial
Specimen Volume: 0.6 mL
Collection Instructions: Within 2 hours of collection, centrifuge and aliquot serum into a plastic vial.

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	7 days	
	Ambient	24 hours	
	Frozen	365 days	

## Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
FERR1	Ferritin, S	Numeric	mcg/L	20567-4

LOINC<sup>®</sup> and CPT codes are provided by the performing laboratory.

#### Supplemental Report:

No

## **CPT Code Information:**

82728

## **Reference Values:**

Males:

0 days-4 weeks: 150-973 mcg/L 5 weeks-5 months: 9-580 mcg/L 6 months-9 years: 6-111 mcg/L 10-17 years: 15-201 mcg/L > or =18 years: 31-409 mcg/L

Females:

0 days-4 weeks: 150-973 mcg/L 5 weeks-5 months: 9-580 mcg/L 6 months-17 years: 8-115 mcg/L 18-50 years: 6-175 mcg/L



# **Test Definition: FERR1**

Ferritin, Serum

> or =51 years: 11-328 mcg/L