

Reporting Title: Rufinamide, S
Performing Location: Rochester

Specimen Requirements:

Supplies: Sarstedt Aliquot Tube, 5 mL (T914)

Collection Container/Tube:

Preferred: Red top

Acceptable: Serum gel

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Collection Instructions:

- 1. Collect blood immediately before next scheduled dose.
- 2. For sustained-release formulations ONLY, collect blood a minimum of 12 hours after last dose.
- 3. Centrifuge and aliquot serum into plastic vial within 2 hours of collection.

Forms:

If not ordering electronically, complete, print, and send 1 of the following forms with the specimen:

- [Therapeutics Test Request](#) (T831)
- [Neurology Specialty Testing Client Test Request](#) (T732)

Specimen Type	Temperature	Time	Special Container
Serum	Ambient	28 days	
	Refrigerated (preferred)	28 days	
	Frozen	28 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
63030	Rufinamide, S	Numeric	mcg/mL	59323-6

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

80210

Reference Values:

5.0-30.0 mcg/mL