

**Reporting Title:** Fentanyl Screen, U  
**Performing Location:** Mayo Clinic Laboratories - Rochester Superior Drive

**Ordering Guidance:**  
For situations where chain of custody is required, a Chain of Custody Kit (T282) is available. For chain-of-custody testing, order FENTX / Fentanyl with Metabolite Confirmation, Chain of Custody, Random, Urine.

For monitoring therapeutic drug levels, order FENTS / Fentanyl, Serum.

**Additional Testing Requirements:**  
If urine creatinine is required or adulteration of the sample is suspected, the following test should also be ordered, ADULT / Adulterants Survey, Random, Urine.

**Specimen Requirements:**  
**Supplies:** Sarstedt Aliquot Tube, 5 mL (T914)  
**Collection Container/Tube:** Clean, plastic urine collection container  
**Submission Container/Tube:** Plastic, 5-mL tube  
**Specimen Volume:** 2 mL  
**Collection Instructions:**  
1. Collect a random urine specimen.  
2. No preservative.

**Forms:**  
If not ordering electronically, complete, print, and send a [Therapeutics Test Request](#) (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	14 days	
	Ambient	72 hours	
	Frozen	14 days	

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
63060	Fentanyl Screen, U	Alphanumeric	ng/mL	59673-4

LOINC® and CPT codes are provided by the performing laboratory.

**Supplemental Report:**  
No

**CPT Code Information:**  
80307

**Reference Values:**

---

Negative

Screening cutoff concentration: 2 ng/mL