



# Test Definition: ETGS

Ethyl Glucuronide Screen, Random, Urine

**Reporting Title:** Ethyl Glucuronide Screen, U

**Performing Location:** Mayo Clinic Laboratories - Rochester Superior Drive

**Ordering Guidance:**

For situations where chain of custody is required, a Chain of Custody Kit (T282) is available. For chain-of-custody testing, order ETGX / Ethyl Glucuronide Confirmation, Chain of Custody, Random, Urine.

**Additional Testing Requirements:**

**Specimen Requirements:**

**Supplies:** Sarstedt Aliquot Tube, 5 mL (T914)

**Collection Container/Tube:** Plastic urine container

**Submission Container/Tube:** Plastic, 5 mL tube

**Specimen Volume:** 2 mL

**Collection Instructions:**

1. Collect a random urine specimen.
2. No preservative.

**Forms:**

If not ordering electronically, complete, print, and send a [Therapeutics Test Request](#) (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	28 days	
	Ambient	28 days	
	Frozen	28 days	

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
63420	Ethyl Glucuronide Screen, U	Alphanumeric	ng/ml	58375-7

LOINC® and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

**CPT Code Information:**

80307

**Reference Values:**

Negative

Screening cutoff concentration: 500 ng/mL