

Reporting Title: Troponin T, 5th gen, P
Performing Location: Mayo Clinic Laboratories - Rochester Main Campus

Specimen Requirements:

Collection Container/Tube:

Preferred: Light-green top (lithium heparin gel)

Acceptable: Green top (lithium heparin)

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Collection Instructions:

- 1. Lithium heparin gel tubes should be centrifuged within 2 hours of collection.
- 2. Plasma from lithium heparin tubes should be centrifuged and aliquoted into a plastic vial within 2 hours of collection.

Forms:

If not ordering electronically, complete, print, and send a [Cardiovascular Test Request Form](#) (T724) with the specimen.

Specimen Type	Temperature	Time	Special Container
Plasma Li Heparin	Frozen (preferred)	365 days	
	Ambient	24 hours	
	Refrigerated	24 hours	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
TRPS	Troponin T, 5th gen, P	Numeric	ng/L	67151-1

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

84484

Reference Values:

Males: < or =15 ng/L

Females: < or =10 ng/L