



# Test Definition: FWARP

Warfarin, Plasma

**Reporting Title:** Warfarin, Plasma

**Performing Location:** NMS Labs

**Specimen Requirements:**

**\*\*Must submit one specimen per order. Specimens cannot be shared between multiple orders.\*\***

**Collection Container/Tube:** EDTA (lavender top) or pink top

**Submission Container/Tube:** Plastic vial

**Specimen Volume:** 1 mL

**Collection Instructions:**

1. Draw blood in EDTA (lavender top) or pink top tube(s). **Plasma gel tube is not acceptable.**
2. Centrifuge and send 1 mL of plasma refrigerate in preservative-free plastic vial.

Specimen Type	Temperature	Time	Special Container
Plasma EDTA	Refrigerated (preferred)	30 days	
	Ambient	30 days	
	Frozen		

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
Z5526	Warfarin	Alphanumeric		4098-0

LOINC® and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

**CPT Code Information:**

80375

**Reference Values:**

Reporting limit determined each analysis.

None Detected mcg/mL