

Reporting Title: Scleroderma Comp Profile

Performing Location: Esoterix Endocrinology

Specimen Requirements:

Collection Container/Tube: 10 mL Red

Submission Container/Tube: Plastic vial

Specimen Volume: 3 mL

Acceptable: SST

Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube is acceptable. Spin down and send 3 mL of serum refrigerated in a plastic vial.

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	14 days	
	Ambient	7 days	
	Frozen	60 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
Z5784	Anti-Nuclear Ab by IFA	Alphanumeric		Not Provided
Z5785	Anti-Centromere Ab	Alphanumeric		Not Provided
Z5786	Anti-RNA Polymerase III	Alphanumeric		Not Provided
Z5787	Anti-Scl-70 Ab	Alphanumeric		Not Provided
Z5788	Anti-U1 RNP Ab	Alphanumeric		Not Provided
Z5789	Anti-U3 RNP (Fibrillarin)	Alphanumeric		Not Provided
Z5790	Anti-Th/To Ab	Alphanumeric		Not Provided
Z5791	Anti-PM/Scl-100 Ab	Alphanumeric		Not Provided

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

86038

83516

86235 x 5

86256

Reflex Tests:

Test Id	Reporting Name	CPT Units	CPT Code	Always Performed	Available Separately
FANTP	ANA Titer and Pattern			No	No

FCA70	Repeat Anti Scl-70	1	86235	No	No
-------	--------------------	---	-------	----	----

Result Codes for Reflex Tests:

Test ID	Result ID	Reporting Name	Type	Unit	LOINC®
FCA70	FCA70	Repeat Anti Scl-70	Alphanumeric		Not Provided
FANTP	Z5792	Homogeneous Pattern	Alphanumeric		Not Provided
FANTP	Z5793	Nucleolar Pattern	Alphanumeric		Not Provided
FANTP	Z5794	Speckled Pattern	Alphanumeric		Not Provided
FANTP	Z5795	Centromere Pattern	Alphanumeric		Not Provided
FANTP	Z5796	Spindle Apparatus Pattern	Alphanumeric		Not Provided
FANTP	Z5797	Nuclear Membrane Pattern	Alphanumeric		Not Provided
FANTP	Z5798	Midbody Pattern	Alphanumeric		Not Provided
FANTP	Z5799	Nuclear Dot Pattern	Alphanumeric		Not Provided
FANTP	Z5800	PCNA Pattern	Alphanumeric		Not Provided
FANTP	Z5801	Centriole Pattern	Alphanumeric		Not Provided
FANTP	Z5802	Note:	Alphanumeric		Not Provided

Reference Values:

Anti-Nuclear Ab by IFA, Anti-U3 RNP(Fibrillarin), Anti-Th/To Ab:

Reference Range: Negative

Anti-Centromere Ab:

Reference Range: <1:40

Anti-SCL-70 Ab:

Reference Range: <20

Anti-RNA Polymerase III, Anti-U1RNP Ab, and Anti-PM/Scl-100 Ab:

Reference Range: <20

Negative: <20 units

Weak Positive: 20-39 units

Moderate Positive: 40-80 units

Strong Positive: >80 units