

Reporting Title: Amylase, Total, S

Performing Location: Mayo Clinic Jacksonville Clinical Lab

Necessary Information:

Patient's age and sex are required.

Specimen Requirements:

Collection Container/Tube:

Preferred: Serum gel

Acceptable: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 0.5 mL

Collection Instructions:

1. Serum gel tubes should be centrifuged within 2 hours of collection.
2. Red-top tubes should be centrifuged, and the serum aliquoted into a plastic vial within 2 hours of collection.

Forms:

Specimen Type	Temperature	Time	Special Container
Serum	Ambient (preferred)	7 days	
	Refrigerated	30 days	
	Frozen	30 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
AMS	Amylase, Total, S	Numeric	U/L	1798-8

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

82150

Reference Values:

0-30 days: 0-6 U/L

31-182 days: 1-17 U/L

183-365 days: 6-44 U/L

1-3 years: 8-79 U/L

4-17 years: 21-110 U/L

> or =18 years: 28-100 U/L

