

# **Test Definition: FOL**

Folate, Serum

### Reporting Title: Folate, S

Performing Location: Mayo Clinic Jacksonville Clinical Lab

#### **Specimen Requirements:**

#### Patient preparation:

1. Patient should be fasting for 8 hours.

2. Do not order on patients who have recently received methotrexate or other folic acid antagonists.

#### **Collection Container/Tube:**

Preferred: Serum gel

Acceptable: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 0.6 mL

#### **Collection Instructions:**

1. Serum gel tubes should be centrifuged within 2 hours of collection.

2. Red-top tubes should be centrifuged, and the serum aliquoted into a plastic vial within 2 hours of collection.

#### Forms:

If not ordering electronically, complete, print, and send a <u>Benign Hematology Test Request Form</u> (T755) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	7 days	
	Frozen	90 days	

## **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
FOL	Folate, S	Numeric	mcg/L	2284-8

LOINC<sup>®</sup> and CPT codes are provided by the performing laboratory.

#### **Supplemental Report:**

No

# **CPT Code Information:**

82746

#### **Reference Values:**

> or =4.0 mcg/L
<4.0 mcg/L suggests folate deficiency</pre>