

Reporting Title: Osteocalcin, S  
Performing Location: Rochester

Specimen Requirements:

Patient Preparation:

1. Fasting (12 hours)
2. For 12 hours before specimen collection do not take multivitamins or dietary supplements containing biotin (vitamin B7), which is commonly found in hair, skin, and nail supplements and multivitamins.

Supplies: Sarstedt Aliquot Tube, 5 mL (T914)

Collection Container/Tube:

Preferred: Serum gel

Acceptable: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Collection Instructions: Centrifuge and aliquot serum into plastic vial.

Specimen Type	Temperature	Time	Special Container
Serum	Ambient	72 hours	
	Refrigerated (preferred)	14 days	
	Frozen	90 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
OSCAL	Osteocalcin, S	Numeric	ng/mL	2697-1

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

83937

Reference Values:

Males

<5 years: 19-75 ng/mL

5-9 years: 21-108 ng/mL

10-15 years: 19-159 ng/mL

16-17 years: 12-114 ng/mL

> or =18 years: 9-42 ng/mL

Females

<5 years: 14-126 ng/mL

---

5-9 years: 16-152 ng/mL  
10-15 years: 15-151 ng/mL  
16-17 years: 9-70 ng/mL  
> or =18 years: 9-42 ng/mL