

Test Definition: CDU

Cadmium, 24 Hour, Urine

Reporting Title: Cadmium, 24 Hr, U

Performing Location: Rochester

Ordering Guidance:

If employees are being monitored in the workplace, OSHA requires that laboratory reports express the cadmium excretion rate per gram of creatinine rather than per 24 hours. Order CDUOE / Cadmium, Occupational Exposure, Random, Urine to accommodate that requirement. Mayo Clinic Laboratories is certified to provide this test.

Necessary Information:

24-Hour volume (in milliliters) is required.

Specimen Requirements:

Patient Preparation:

- 1. For the 48-hour period prior to start of collection, as well as during the collection, patient **should not** eat seafood.
- 2. High concentrations of gadolinium and iodine are known to potentially interfere with most inductively coupled plasma mass spectrometry-based metal tests. If either gadolinium or iodine containing contrast media has been administered, a specimen should not be collected for 96 hours.

Supplies: Urine Tubes, 10 mL (T068)

Collection Container/Tube: Clean, plastic urine container with no metal cap or glued insert

Submission Container/Tube: Plastic, 10-mL urine tube or clean, plastic aliquot container with no metal cap or glued

insert

Specimen Volume: 3 mL **Collection Instructions:**

- 1. Collect urine for 24 hours.
- 2. Refrigerate specimen within 4 hours of completion of 24-hour collection.
- 3. See Metals Analysis Specimen Collection and Transport for complete instructions.

Additional Information: See <u>Urine Preservatives-Collection and Transportation for 24-Hour Urine Specimens</u> for multiple collections.

Urine Preservative Collection Options:

Note: The addition of preservative or application of temperature controls **must occur within 4 hours of completion** of the collection.

Ambient (no	OK	
additive)		
Refrigerate (no	Preferred	
additive)		
Frozen (no additive)	ОК	
50% Acetic Acid	ОК	
Boric Acid	No	
Diazolidinyl Urea	No	
6M Hydrochloric	ОК	
Acid		
6M Nitric Acid	ОК	
Sodium Carbonate	No	
Thymol	No	
Toluene	No	



Test Definition: CDU

Cadmium, 24 Hour, Urine

Specimen Type	Temperature	Time	Special Container
Urine	Ambient	28 days	
Refrigerated (preferred)		28 days	
	Frozen	28 days	

Ask at Order Entry (AOE) Questions:

Test ID	Question ID	Description	Туре	Reportable
CDU	TIME7	Collection Duration (h)	Plain Text	Yes
CDU	VL83	Volume (mL)	Plain Text	Yes

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
31106	Cadmium, 24 Hr, U	Numeric	mcg/24 h	5612-7
TIME7	Collection Duration (h)	Alphanumeric		13362-9
VL83	Volume (mL)	Alphanumeric		3167-4

 $\ensuremath{\mathsf{LOINC}}\xspace^{\ensuremath{\texttt{@}}}$ and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

82300

Reference Values:

0-17 years: Not established > or =18 years: <0.7 mcg/24 h