

**Reporting Title:** Posaconazole, S  
**Performing Location:** Mayo Clinic Laboratories - Rochester Superior Drive

**Specimen Requirements:**  
**Supplies:** Sarstedt Aliquot Tube, 5 mL (T914)  
**Collection Container/Tube:** Red top (serum gel/SST are **not acceptable**)  
**Submission Container/Tube:** Plastic vial  
**Specimen Volume:** 2 mL serum  
**Collection Instructions:** Within 2 hours of collection, centrifuge, and aliquot serum into plastic vial.

**Forms:**  
If not ordering electronically, complete, print, and send a [Therapeutics Test Request](#) (T831) with the specimen.

| Specimen Type | Temperature              | Time    | Special Container |
|---------------|--------------------------|---------|-------------------|
| Serum Red     | Refrigerated (preferred) | 28 days |                   |
|               | Ambient                  | 28 days |                   |
|               | Frozen                   | 28 days |                   |

**Result Codes:**

| Result ID | Reporting Name  | Type    | Unit  | LOINC®  |
|-----------|-----------------|---------|-------|---------|
| 89591     | Posaconazole, S | Numeric | ng/mL | 53731-6 |

LOINC® and CPT codes are provided by the performing laboratory.

**Supplemental Report:**  
No

**CPT Code Information:**  
80187

**Reference Values:**  
>700 ng/mL (trough)