

**Overview**
**Method Name**

ImmunoCAP FEIA

**NY State Available**

Yes

**Specimen**
**Specimen Type**

Serum

**Specimen Required**

Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.

**Specimen Minimum Volume**

0.5 mL

**Reject Due To**

Hemolysis	NA
Lipemia	Mild OK; Gross reject
Icterus	NA
Other	NA

**Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Frozen	365 days	
	Ambient	28 days	

**Clinical and Interpretive**
**Reference Values**

&lt;0.35 kU/L

**Interpretation**

Class	IgE (kU/L)	Comment
0	<0.10	Negative
0/1	0.10 - 0.34	Equivocal/Borderline

1	0.35 - 0.69	Low Positive
2	0.70 - 3.49	Moderate Positive
3	3.5 - 17.49	High Positive
4	17.5 - 49.99	Very High Positive
5	50.0 - 99.99	Very High Positive
6	> 99.99	Very High Positive

## Performance

### Method Description

The test method is the Phadia ImmunoCAP allergen-specific IgE system.

### PDF Report

No

### Day(s) and Time(s) Test Performed

Monday through Friday

### Analytic Time

1 - 2 days

### Maximum Laboratory Time

3 - 6 days

### Performing Laboratory Location

Viracor Eurofins

## Fees and Codes

### Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their Regional Manager. For assistance, contact [Customer Service](#).

### Test Classification

This test was developed and its performance characteristics determined by Viracor Eurofins. It has not been cleared or approved by the U.S. Food and Drug Administration.

### CPT Code Information

86003

### LOINC® Information



---

Test ID	Test Order Name	Order LOINC Value
FBLME	Bean Lima IgE	7131-6

Result ID	Test Result Name	Result LOINC Value
Z3080	Bean Lima IgE	7131-6
Z3081	Class	15566-3