

Overview**Method Name**

Enzyme Immunoassay (FEIA)

NY State Available

No

Specimen**Specimen Type**

Serum

Specimen Required

Specimen Type: Serum

Container/Tube: Red or SST

Specimen Volume: 0.5 mL

Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.

Specimen Minimum Volume

0.5 mL

Reject Due To

Gross hemolysis:	Mild OK; Gross Reject
Thawing:	Warm OK; Cold OK
Gross lipemia:	Reject
Gross icterus:	NA
Other:	NA

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Frozen	365 days	
	Ambient	7 days	

Clinical and Interpretive**Reference Values**

Reference ranges have not been established for food-specific IgG tests. The clinical utility of food-specific IgG tests has not been established. These tests can be used in special clinical situations to select foods for evaluation by diet elimination and challenge in patients who have food-related complaints. It should be recognized that the presence of food-specific IgG alone cannot be taken as evidence of food allergy and only indicates immunologic sensitization by the food allergen in question. This test should only be ordered by physicians who recognize the limitations of the test.

Performance

Method Description

Enzyme immunoassay (FEIA). A standard curve is used to calculate the specific IgG concentrations. The calibrators are referenced to the International Reference Preparation for serum immunoglobulins. This test has not been cleared or approved for diagnostic use by the U.S. Food and Drug Administration.

PDF Report

No

Day(s) and Time(s) Test Performed

Monday through Friday

Analytic Time

3 days

Maximum Laboratory Time

5 - 7 days

Performing Laboratory Location

Viracor Eurofins

Fees and Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their Regional Manager. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by Viracor Eurofins. It has not been cleared or approved by the U.S. Food and Drug Administration.

CPT Code Information

86001

LOINC® Information

Test ID	Test Order Name	Order LOINC Value
FRFYG	Rye Food IgG	51933-0



Result ID	Test Result Name	Result LOINC Value
FRFYG	Rye Food IgG	51933-0