

Overview
Reflex Tests

| Test ID | Reporting Name | Available Separately | Always Performed |
|---------|---------------------|----------------------|------------------|
| FOVAT | Anti-Ovary Ab Titer | No | No |

Testing Algorithm

If FOVAS "Ovarian Ab Screen w/Reflex" is positive, then FOVAT "Anti-Ovary Ab Titer" will be performed at an additional charge.

Method Name

Immunofluorescence Assay (IFA)

NY State Available

Yes

Specimen
Specimen Type

Serum

Specimen Required

Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 1 mL of serum refrigerated in a plastic vial.

Specimen Minimum Volume

0.3 mL

Reject Due To

| | |
|-----------|-----------------------|
| Hemolysis | Mild OK; Gross reject |
| Lipemia | Mild OK; Gross reject |
| Icterus | NA |
| Other | NA |

Specimen Stability Information

| Specimen Type | Temperature | Time | Special Container |
|---------------|--------------------------|---------|-------------------|
| Serum | Refrigerated (preferred) | 14 days | |
| | Frozen | 30 days | |
| | Ambient | 7 days | |

Clinical and Interpretive**Reference Values**

Negative

Ovarian antibody screen is at 1:5 dilution

Performance**PDF Report**

No

Day(s) and Time(s) Test Performed

Wednesday

Analytic Time

2 days

Maximum Laboratory Time

4 - 17 days

Performing Laboratory Location

Quest Diagnostics Nichols Institute

Fees and Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their Regional Manager. For assistance, contact [Customer Service](#).

CPT Code Information

86255

86256 (if appropriate)

LOINC® Information

| Test ID | Test Order Name | Order LOINC Value |
|---------|----------------------------|-------------------|
| FOVAS | Ovarian Ab Screen w/Reflex | 21436-1 |

| Result ID | Test Result Name | Result LOINC Value |
|-----------|---------------------|--------------------|
| Z3669 | Anti-Ovary Antibody | 21436-1 |