

Overview**Method Name**

ELISA

NY State Available

Yes

Specimen**Specimen Type**

Serum

Specimen Required

Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 1 mL of serum ambient in a plastic vial.

Reject Due To

| | |
|-----------|-----------------------|
| Hemolysis | Mild OK: Gross Reject |
| Lipemia | Mild OK; Gross reject |
| Icterus | Mild OK: Gross Reject |
| Other | NA |

Specimen Stability Information

| Specimen Type | Temperature | Time | Special Container |
|---------------|---------------------|---------|-------------------|
| Serum | Ambient (preferred) | 7 days | |
| | Frozen | 28 days | |
| | Refrigerated | 7 days | |

Clinical and Interpretive**Reference Values**

Normal

This ELISA measures IgG antibodies specific for IgE. A result of normal indicates that the level of IgG anti-IgE antibodies is similar to that seen in a population of healthy individuals. A result of elevated indicates an increased level of IgG anti-IgE antibodies compared to healthy individuals. These autoantibodies have been implicated as a causative agent in autoimmune chronic urticaria and atopic dermatitis.

Performance

PDF Report

No

Day(s) and Time(s) Test Performed

Set up Wednesday, Report Thursday

Analytic Time

1 - 7 days

Maximum Laboratory Time

7 - 11 days

Performing Laboratory Location

Eurofins Viracor

Fees and Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their Regional Manager. For assistance, contact [Customer Service](#).

CPT Code Information

83516

LOINC® Information

| Test ID | Test Order Name | Order LOINC Value |
|---------|-----------------|-------------------|
| FANTI | Anti-IgE | Not Provided |

| Result ID | Test Result Name | Result LOINC Value |
|-----------|------------------|--------------------|
| FANTI | Anti-IgE | Not Provided |