

**Overview**
**Method Name**

Enzyme Immunoassay (EIA)

**NY State Available**

Yes

**Specimen**
**Specimen Type**

Serum

**Specimen Required**

Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.

**Specimen Minimum Volume**

0.5 mL

**Reject Due To**

Hemolysis:	Mild OK; Gross reject
Thawing:	Warm OK; Cold OK
Lipemia:	Mild OK; Gross reject
Icterus:	Mild OK; Gross reject
Other:	NA

**Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Frozen	365 days	
	Ambient	28 days	

**Clinical and Interpretive**
**Reference Values**

&lt;0.35 kU/L

**Interpretation**

Class	IgE (kU/L)	Comment
0	<0.35	Below Detection

1	0.35 – 0.69	Low Positive
2	0.70 – 3.49	Moderate Positive
3	3.50 – 17.49	Positive
4	17.50 – 49.99	Strong Positive
5	50.00 – 99.99	Very Strong Positive
6	>99.99	Very Strong Positive

## Performance

### Method Description

The conventional EIA uses allergen-coated discs from several suppliers and an enzyme-labeled anti-IgE.

### PDF Report

No

### Day(s) and Time(s) Test Performed

Monday through Friday

### Analytic Time

2 - 3 days

### Maximum Laboratory Time

4 - 7 days

### Performing Laboratory Location

Viracor Eurofins

## Fees and Codes

### Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their Regional Manager. For assistance, contact [Customer Service](#).

### Test Classification

This test was developed and its performance characteristics determined by Viracor Eurofins. It has not been cleared or approved by the U.S. Food and Drug Administration.

### CPT Code Information

86003

### LOINC® Information



---

Test ID	Test Order Name	Order LOINC Value
FRUB	Rhubarb IgE	7667-9

Result ID	Test Result Name	Result LOINC Value
Z3878	Rhubarb IgE	7667-9
Z3879	CLASS	15993-9